



# POINT- OF-CARE



## OSOM® H. pylori Test

### FOR THE DETECTION OF Anti-H. pylori IgG ANTIBODIES

#### Features and Benefits

- **Accurate** – High Sensitivity/Specificity
- **Fast** and easy work flow
- **CLIA Waived** – For finger stick whole blood
- **Exceptional Customer Service** – Online training modules and technical support staffed by certified medical technologists/professionals

#### Performance Characteristics

Sensitivity/Specificity	Sample Type	Storage Temperature	Time to Result
95.9%/89.1%*	Serum, plasma, whole blood**	2-30 °C/36-86°F	10 minutes

\*Compared to biopsy/histology

\*\*CLIA Waived for finger stick whole blood




# OSOM<sup>®</sup> H. pylori Test




Henry P. Lori

## SIMPLE TO USE / RESULTS IN MINUTES


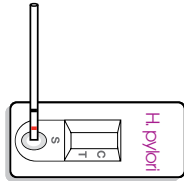


**1** FOR SERUM OR PLASMA, FILL CAPILLARY TUBE TO THE RED LINE. FOR WHOLE BLOOD, FILL TO THE BLACK LINE. ADD SAMPLE BY LIGHTLY TAPPING CAPILLARY TUBE ON UPPER AREA OF THE SAMPLE WELL.

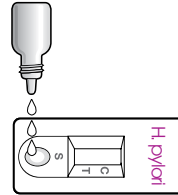


**2** ADD 3 DROPS OF DEVELOPER TO LOWER AREA OF SAMPLE WELL

**3** READ RESULTS AT 10 MINUTES

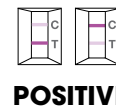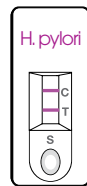

**POSITIVE**

**NEGATIVE**

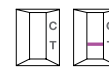

**INVALID**

### Ordering Information

Description	Configuration	Catalog Number	Storage	CPT Code
OSOM <sup>®</sup> H. PYLORI TEST	25 Tests	175 (CE marked)	Room Temp	86318QW (Whole Blood)
H. PYLORI CONTROL KIT	2 X 1mL vials (Pos & Neg)	176	Refrigerated	86677 (Serum, Plasma)

To learn more about all of our OSOM Tests scan the code or go to [osomtests.com](http://osomtests.com)



#### THE AMERICAS

SEKISUI Diagnostics, LLC  
One Wall Street  
Burlington, MA 01803  
Phone: 800 332 1042  
Fax: 800 762 6311  
Email: [questions@sekisui-dx.com](mailto:questions@sekisui-dx.com)

#### INTERNATIONAL

SEKISUI Diagnostics (UK) Limited  
Liphook Way, Allington  
Maidstone, Kent, ME16 0LQ, UK  
Phone: +44 1622 607800  
Fax: +44 1622 607801  
Email: [info@sekisui-dx.com](mailto:info@sekisui-dx.com)

**SEKISUI**  
DIAGNOSTICS

Because every result matters™

[sekisuidiagnostics.com](http://sekisuidiagnostics.com)

