



POINT- OF-CARE



OSOM® iFOB Rapid Test

FOR THE DETECTION OF FECAL OCCULT BLOOD/HUMAN HEMOGLOBIN

Features and Benefits

- **Accurate** – Specific to human hemoglobin
- **Fast** – Results in 5-10 minutes
- **Simple** – Easy three step procedure with <1 minute hands on time
- **Improved Patient Compliance** – No dietary or drug restrictions
- **Exceptional Customer Service** – Online training modules and technical support staffed by certified medical technologists/professionals

Performance Characteristics


Limit of detection	Sample Type	Specimen Stability	Storage Temperature	Time to Result
50µg hHB/g feces	Stool	10 Days*	2-30°C/36-87°F	5 minutes

*Specimen stability after extraction




OSOM® iFOB Rapid Test


SIMPLE TO USE / RESULTS IN MINUTES

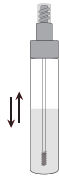

1 SHAKE STOOL COLLECTION TUBE FOR 20-30 SECONDS. TAP TUBE ON HARD SURFACE TO DISLodge TRAPPED AIR

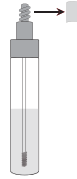


2 UNSCREW TIP CAP ON THE STOOL COLLECTION TUBE

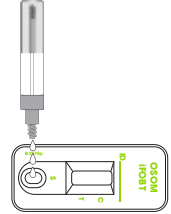

3 SQUEEZE TUBE GENTLY TO DISPENSE THREE (3) DROPS INTO THE SAMPLE WELL




4 READ THE RESULTS IN 5-10 MINUTES

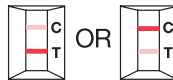

POSITIVE



NEGATIVE

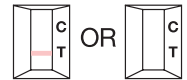

INVALID

Ordering Information

Description	Configuration	Catalog Number	CPT Code
OSOM IFOB TEST	25 Test plus two for QC	1002	Diagnostic 82274QW Screening G0328QW
OSOM IFOB CONTROLS**	1x1ml Pos vial, 1x1ml Neg vial	1000	
OSOM IFOB MAILERS***	25 Mailers/box	1003	
OSOM IFOB TUBES	25 Tubes/box	1000	

**Not included in test kit

***Included in test kit but may be ordered separately

To learn more about all of our OSOM Tests scan the code or go to osomtests.com



THE AMERICAS

SEKISUI Diagnostics,
One Wall Street,
Burlington, MA 01803
Phone: 800 332 1042
Fax: 800 762 6311

Email: questions@sekisui-dx.com

INTERNATIONAL

SEKISUI Diagnostics (UK) Limited,
Liphook Way, Allington
Maidstone, Kent, ME16 0LQ, UK
Phone: +44 1622 607800
Fax: +44 1622 607801

Email: info@sekisui-dx.com

SEKISUI
DIAGNOSTICS

Because every result matters™

sekisuidiagnostics.com

