

dima Print materials

Cleaning and post-curing times

Material	Indication	Colour	Cleaning time in isopropanol (cara Print Clean)	Post-curing time (HiLite Power 3D)
dima Print Cast ruby	CAD-to-cast structures for frameworks, C&B (up to 5 units) and press ceramic	red	pre-cleaning: 1 minutes postcleaning: 1 minutes	3 minutes (1,5 minutes front side + 1,5 minutes reverse side)
dima Print Splint clear	Splints	transparent clear	pre-cleaning: 3 minutes postcleaning: 2 minutes	6 minutes (3 minutes front side + 3 minutes reverse side)
dima Print Stone beige	Dental models	gypsum beige	pre-cleaning: 3 minutes postcleaning: 2 minutes	10 minutes (5 minutes front side + 5 minutes reverse side)
dima Print Stone teal	Orthodontic models	turquoise		
dima Print Gingiva Mask	Gingiva masks	pink		
dima Print Ortho	Splints	transparent light blue		
dima Print Impression Blue	Impression trays	opaque blue		
dima Print Guide	Surgical guides	transparent light orange		
dima Print Cast	CAD-to-cast structures for partial frameworks	raspberry		

Please take notice of the Instructions for Use for each material.



See how cara Print System works:
kulzer.com/cara-print-application

Contact in Germany

Kulzer GmbH
Leipziger Straße 2
63450 Hanau, Germany
cara-service@kulzer-dental.com
kulzer.com/cara-print

Giving a hand to oral health.



KULZER
MITSUI CHEMICALS GROUP